

topshelforthopedics.com
Phone: 866.592.0488 • Fax: 209.834.8832
Tracy, CA 95304
1851 East Paradise Road, Suite A



BOA Lite

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.









BOA Lite

Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of the brace.

Indications: Acute and chronic low back pain, post-operative support, activity-related low back pain.

Contraindications: Pregnancy and circulation, pulmonary cardiovascular or skeletal conditions which have the risk to be made worse as a result of compression and/or pressure.

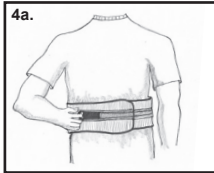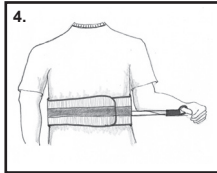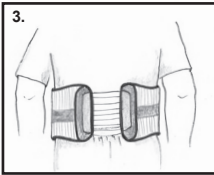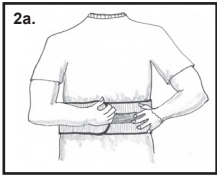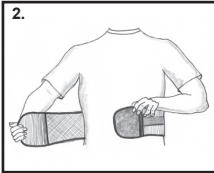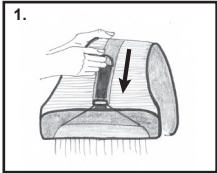

1. Position and fasten the pull-handle as close as possible to the compression panels and fully extend the cord.
2. Referencing the attached label for correct brace orientation, position the brace around the waist and fasten. The pull-handle will be on the left side. The soft lining will be against the body (Fig. 2a).
3. For proper fit and compression, center the spine between the two side panels and position symmetrically on waist.
4. To apply compression, grasp the pull-handle and pull to the right. After the desired level of compression is achieved fasten the pull-handle to the contact strip on the side panels (Fig. 4a).

Use & Care: Hand wash in cold water using mild soap, rinse thoroughly. Air dry only.
Note: If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.

Warning: This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Warranty: Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.

Caution: Federal Law (USA) restricts this device for sale by or on the order of a licensed healthcare professional.