

Phone: 866.592.0488 • Fax: 209.834.8832
Tracy, CA 95304
1851 East Paradise Road, Suite A



High Hand Universal Arm Sling

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.







High Hand Universal Arm Sling

Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

Indications: Post-injury/surgery of the hand and/or wrist requiring elevation.

Application Instructions:

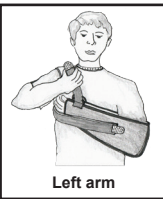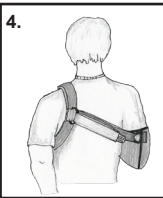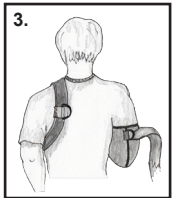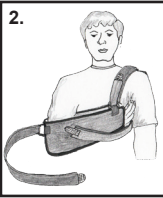

1. Insert arm into sling pocket so that the wide portion is towards the elbow. Wrap the front strap over the shoulder and back under the arm and affix the other end to the front of the sling pocket. Trim length as necessary once the sling is fitted (image 1 & 2).
2. The "D" ring attached with the gray hook material should be positioned directly over the shoulder blade on the back of your shoulder facing out. If it is not correctly positioned, have another person assist in relocating it to the proper location before proceeding (image 3).
3. With the assistance of another person thread the strap that is sewn to the sling pocket by the elbow through the gray "D" ring assembly that is over your shoulder blade. The distal end of this strap has a gray hook tab that will affix itself to the returning portion of the strap. This can be trimmed to length as necessary (image 4).
4. Adjust the height and position the sling to your comfort.
5. **REMOVAL**-To remove the sling release the strap that is secured to the front of the sling pocket. Remove the arm from the pocket. Lift the assembly over the head so that the strap across the back remains secured.
NOTE: LEFT ARM CONVERSION - Gray "D" ring and gray velcro will need to be reversed on the straps.

Use and Care:

Hand wash the liner with mild soap and water and lay out to dry. Do NOT machine dry.

Notes:

For single patient use only.

Warning:

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.